


ERBE VIO/APC2 INSTALLATION DRAWING

SERVICE MODULE TS-723-2

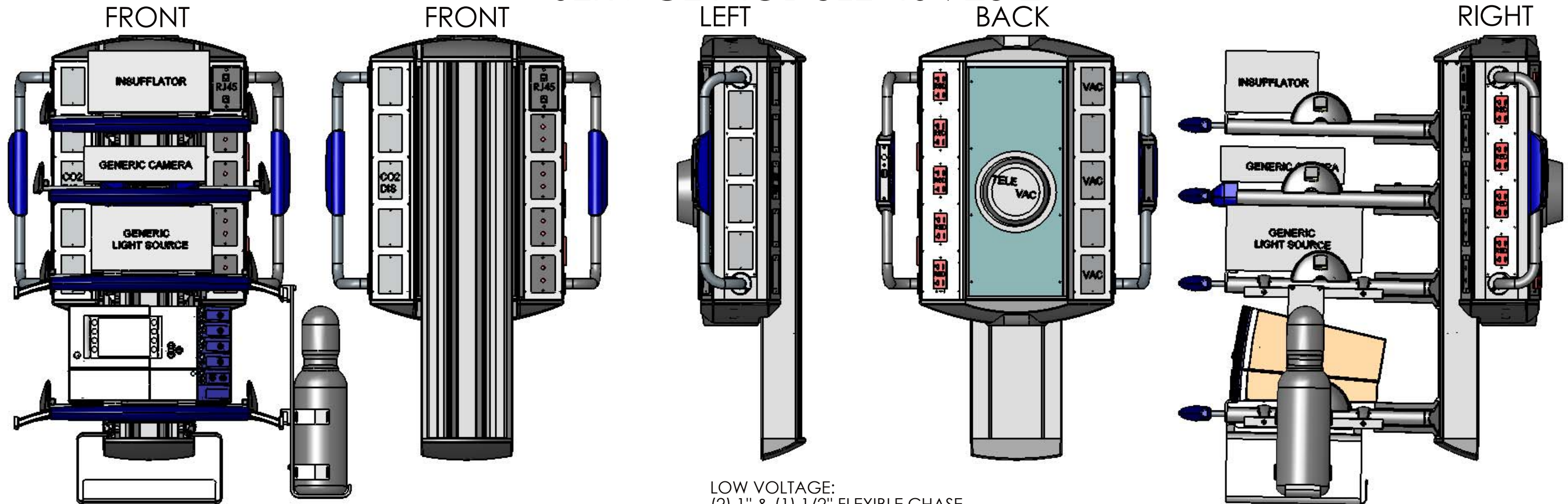

SEE TELETOM PREINSTALL GUIDE FOR ALL NECESSARY PRE-INSTALLATION INFORMATION

ELECTRICAL CIRCUITS:
6 CIRCUITS:

- (3) 20A CIRCUITS FOR EMERGENCY RECEPTACLES
- (1) 20A CIRCUIT FOR TELEVAC SMOKE EVACUATION SYSTEM
- (1) 20A CIRCUIT FOR RECEPTACLES IN PLATFORMS
- (1) 20A CIRCUIT FOR TELETOM MOTOR AND ELECTROMECHANICAL BRAKE

ELECTRICAL RECEPTACLES:

- (9) RED DUPLEX RECEPTACLES(EMERGENCY), 20A
 - (6) SIMPLEX RECEPTACLES IN PLATFORMS (EMERGENCY) 20A (6/CIRCUIT)
- Power Source: Not Provided

GASES THAT NEED TO BE PIPED IN:

- (3) VAC TBD
- (1) CO2 D.I.S.S.
- (1) N2 FOR PNEUMATIC BRAKES, LINE PRESSURE (NON-REGULATED)
- (10) BLANKS

LOW VOLTAGE:

- (2) 1" & (1) 1/2" FLEXIBLE CHASE
- (4) DOUBLE GROMMET PLATE
- (1) Cat 5 RJ-45 PLATE WITH 2 OUTLETS

ACCESSORIES:

- (SEPARATE LINE ITEM - NOT INCLUDED IN TELETOM PRICING)
- TELEVAC SMOKE EVACUATION SYSTEM
- TANK HOLDER

SUPPORTED EQUIPMENT: (CUSTOMER PROVIDED)

ERBE VIO / APC 2 SYSTEM

BRAKE SYSTEMS:

- ELECTROMECHANICAL BRAKES
- PNEUMATIC BRAKES

PLATFORMS:

- (3) CONFORM PLATFORMS
- (1) CONFORM CONTROL PLATFORMS
- (1) 950MM EQUIPMENT COLUMN

WEIGHT: 801 LBS

MOMENT: 4071 FT LBS

ELEVATION REFERENCE:N/A

I confirm the gas fitting type and the configuration of the gases, electric, low voltage provisions, and platforms and understand that any changes made after an order is in production will result in a change order fee and a shipment delay.		PROJECT: BERCHTOLD CORPORATION USA NORTH CHARLESTON,SC ERBE BOOM	BERCHTOLD Corp. 1950 Hanahan Rd. N. Charleston, SC 29406 (800)243-5135
Authorized Signature:	SCALE: 1:12	QUANTITY:	
Title: _____ Date: _____	FILENAME: ERBE.Service Module Matrix-with Endo	SALES MARKETING	
Delivery Date Required:	DRAWN BY: FTA	DATE: 5/31/2005	

REV	DATE:	REV BY:	DESCRIPTION:
A	8/5/2005	FTA	ADDED IN ARGON TANK MODEL
-	-	-	-
-	-	-	-

