


ERBEFLO™ SET UP INSTRUCTIONS

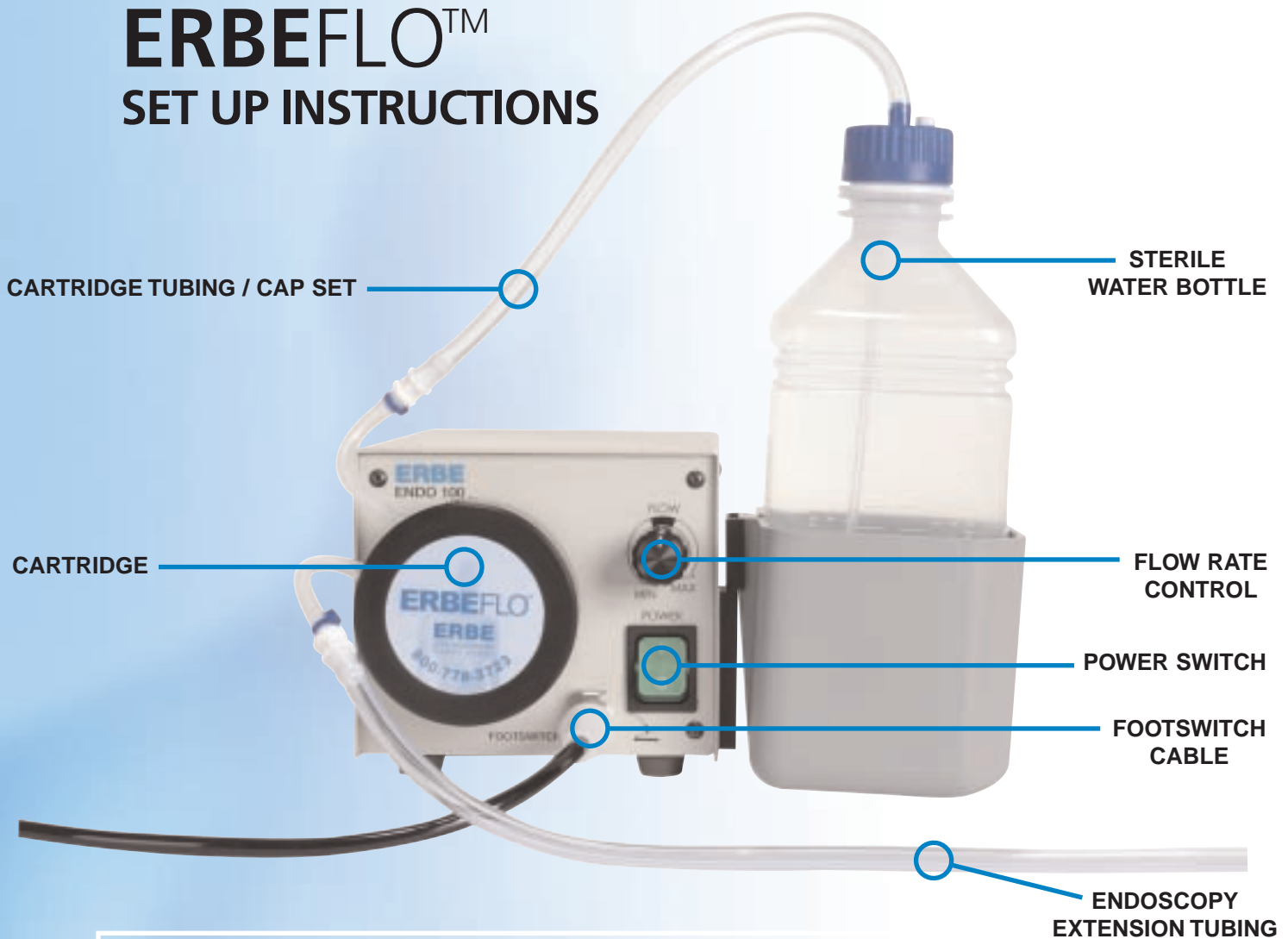

ERBEFLO™ OPERATING STEPS

- **Open Packages.** Remove the Lavage Pump Cartridge Tubing/Cap Set.
- **Open Your Sterile Water Bottle.** Place bottle tubing with the weight of the Set into the sterile water bottle. Tighten the cap.
- **Load the Cartridge.** Place the white tubing ring into the reusable cartridge, with the tubing in the correct slots.
- **Lock the Cartridge.** Line up the silver bar closest to the tubing with the top of the pump housing. Slide the cartridge on and lock it into place.
- **Connect the Cartridge Tubing.** Connect the cartridge tubing to the Endoscopy Channel Tubing* or scope specific Extension Tubing.
- **Footswitch.** Plug in the Footswitch cable.
- **Set Flowrate Control.** Set rate in midrange, white marker straight up.
Clock-wise turn increases flow rate.
- **Powerswitch.** Turn to ON position. Green light shows power is ON.
- **Prime the Pump.** Depress the Footswitch for about 5 seconds to remove air and fill tubing with sterile water. ERBEFLO is now **ready for use**. Adjust flow as needed.

24 HOUR USE FOR TUBING/CAP SET AND SCOPE SPECIFIC EXTENSION TUBING.
ENDOSCOPY CHANNEL TUBING*, IF USED, IS SINGLE USE ONLY.

**FOR CUSTOMER SUPPORT
CONTACT 800.778.3723**

www.erbe-usa.com

*Pending FDA Approval

LIT/5088/00 01/04

ERBE
USA INCORPORATED
Surgical Systems

ERBEFLO™

Accessories for the ERBE Model
ENDO 100™ Lavage Pump

MODEL ENDO 100 LAVAGE PUMP, FOOTSWITCH AND MOUNTING BRACKET

MODEL #



ENDO 100' Lavage Pump (System includes Standard Reusable Accessories) *Note: Only picture of the Pump unit shown.*

7910-3000



Footswitch, Single, Pneumatic

7910-3002



Mounting Bracket for ESU Models ICC 200, 300, 350

20128-007

MODEL ENDO 100 LAVAGE PUMP REUSABLE SYSTEM ACCESSORIES



Cartridge with Tubing

7910-3001



Extension Tubing (disposable, 12/package)

7910-3003



Irrigation Bottle with Bracket, 500ml

7910-3004



Channel Adapter

7910-3005

MODEL ENDO 100 LAVAGE PUMP 24 HOUR USE DISPOSABLE SYSTEM ACCESSORIES



Lavage Pump Cartridge (reusable)

7911-3001



Lavage Pump Cartridge Tubing/Cap Set (24 hour use)

7911-3002



Water Bottle Holder for Pump Model ENDO 100 (reusable)

7911-3003



Endoscopy Channel Tubing* (single use)

7911-3004



Endoscopy Extension Tubing for Olympus Model 160 Series Scopes (24 hour use)

7911-3005

**Pending FDA Approval*

ERBE
USA INCORPORATED
Surgical Systems

2225 Northwest Parkway
Marietta, GA 30067

Tel. 800. 778. 3723
Tel. 770. 955. 4400
Fax. 770. 955. 2577